

radica™

provisional & diagnostic resin

CHAIRSIDE GUIDE

DENTSPLY
CERAMCO

The chemistry of a great smile.™

PLACEMENT TECHNIQUE



1 Test fit of the restoration onto finished tooth preparations. Evaluate and adjust for complete seating with acceptable occlusion.



2 Clean the internal surface with aluminum oxide or lightly grind with 1/4 round bur.



3 Refine and replace restoration with an MMA acrylic such as Biolon,® a bis-GMA such as Integrity™ Temporary C&B Material or Temporary Bridge Resin from Caulk. Consult manufacturer's instructions for proper relining technique.



4 Reseat the case and allow excess material to express.



5 Remove excess material.



6 Upon removal of the restoration, finish and polish margins.



7 Apply a temporary cement and seat restoration.



8 Finish the case and polish with a product such as Enhance® Finishing System or Po-60® One Step Diamond Micro-Polisher.