



Care Guide:  
**IMPACT Dentures**

## Cleaning Instructions



1

Rinse denture with warm, soapy water



2

Remove debris with soft brush



3

Soak in cleaner for 3 to 5 minutes

1. Remove your denture and rinse with warm, soapy water (**not to exceed 113°F / 45°C**) to remove food debris or other loose particles.
2. Use a soft brush to remove debris from the denture.
3. Using a mild, non-abrasive cleaning agent approved for dentures, soak for three to five minutes. **Do not use mouthwash or toothpaste on the denture. Do not soak in cleaning solution overnight.**

## Recommendations

- We recommend cleaning your dentures daily, preferably after every meal.
- A few prescription medications may cause some discoloration or damage to the dentures.



4

Brush and rinse mouth



5

Rinse denture with warm or cold water



6

If not using, let soak in water

4. Use a soft brush to clean your gums, tongue, cheeks, and roof of your mouth. If used, remove any remaining denture adhesive from your gums.
5. Rinse dentures with water before putting them back in your mouth.
6. If not putting in your mouth, the dentures can be left in a tray or cup with some water overnight to keep them from drying out. **Only soak overnight in water. Do not soak in mouthwash or strong solutions.**

- Smoking and chewing tobacco can have an adverse effect on the finish of the dentures.
- Ask your dentist about your no-appointment denture reorder process.