

	G-CAM / TLZ-IB with Metal Bar		TLZ-IB	
	Coping-Free	With iJIG	With Verified Model	
<b>Appointment 1:</b> Post-operative healing records	<ul style="list-style-type: none"> <li>Scan upper, lower, bite, and surrounding tissue</li> <li>Remove prosthesis and scan healed tissue and MUA's</li> <li>Scan seated ArchBridge scan bodies</li> <li>Doctor sends records to ROE</li> <li><i>Optional:</i> ROE designs Printed Try-In as a prototype for the final prosthesis</li> </ul>	<ul style="list-style-type: none"> <li>Scan upper, lower, bite, and surrounding tissue</li> <li>Remove prosthesis and scan healed tissue and MUA's</li> <li>Scan seated ArchBridge scan bodies</li> <li>Doctor sends records to ROE</li> <li><i>Optional:</i> ROE designs Printed Try-In as a prototype for the final, and iJIG as verification jig</li> </ul>	<ul style="list-style-type: none"> <li>Scan upper, lower, bite, and surrounding tissue</li> <li>Remove prosthesis and scan healed tissue and MUA's</li> <li>Scan seated ArchBridge scan bodies</li> <li>Screw MUA analogs into the prosthesis and set in stone for verified model creation. Ensure stone is NOT touching the prosthesis - it should be 3/4 way up the model analogs</li> <li>Doctor sends records to ROE</li> </ul>	
<b>Appointment 2:</b> Printed Try-In records	<ul style="list-style-type: none"> <li>Doctor seats Printed Try-In and makes any necessary adjustments</li> <li>Scan upper, lower, bite, and surrounding tissue</li> <li><i>Optional:</i> If additional tissue adaptation is needed, remove prosthesis and scan healed tissue and MUA's</li> <li>Doctor sends records to ROE</li> <li>ROE creates final prosthesis and mails to doctor</li> </ul>	<ul style="list-style-type: none"> <li>Doctor seats Printed Try-In and makes any necessary adjustments</li> <li>Scan upper, lower, bite, and surrounding tissue</li> <li><i>Optional:</i> If additional tissue adaptation is needed, remove prosthesis and scan healed tissue and MUA's</li> <li>Seat iJIG and lute to confirm passivity</li> <li>Doctor sends records (with luted iJIG) to ROE</li> <li>ROE creates final prosthesis and mails to doctor</li> </ul>	<ul style="list-style-type: none"> <li>Doctor seats Printed Try-In and makes any necessary adjustments</li> <li>Doctor sends Printed Try-In to ROE</li> <li>ROE creates final prosthesis and mails to doctor</li> </ul>	
<b>Appointment 3:</b> Final seating	Doctor seats final prosthesis	Doctor seats final prosthesis	Doctor seats final prosthesis	

**Pricing** (per arch)

**G-Cam: \$1,130/arch**  
*(includes Printed Try-In and final prosthesis)*

**\$2,555/arch**  
*(includes iJIG, Printed Try-In, and final prosthesis)*

**\$2,430/arch**  
*(includes Printed Try-In and final prosthesis)*

**TLZ-IB with Metal Bar: \$3,288/arch**  
*(includes Printed Try-In and final prosthesis with bar)*

In partnership with:



To learn more, visit [www.roedentallab.com/NADG](http://www.roedentallab.com/NADG)

