

All-Acrylic Partial or Unilateral (Nesbit) Partial

Overview

Acrylic partial dentures offer a budget-friendly solution that can be easily modified and repaired. While not as sturdy as other materials, they are lightweight and can work well as a temporary solution or for patients who need a flexible treatment option. These dentures can be adjusted easily and are typically more comfortable at first, though they may need more frequent maintenance over time.

With all-acrylic and unilateral (Nesbit) partial dentures, simply capture records and seat on the next appointment.

TECHNOLOGY & MATERIALS

- Optional:* IOS scanner
- Optional:* PVS impression material
- Pressure indicating paste
- Denture polishing and finishing tools

All-Acrylic Partial Workflow

1. Take the following photos with the patient standing up:

- Full-face full-smile
- Full-face exaggerated smile
- *Optional:* Full-face profile



2. *Optional:* If the patient has an existing prosthesis, seat before capturing records. Capture records including:

- Capture IOS scan of upper, lower, and bite, OR
- Capture physical impression and bite



-
3. Send your records to ROE and complete the Rx via www.roedentallab.com/upload. If using an IOS, submit your scans via the IOS portal, OR export your .STLs and upload to the ROE portal.



-
4. Use pressure indicating paste to determine pressure areas. Make adjustments as necessary, finish, and polish.



-
5. Seat the new final partial denture.

